


# GAINSBOROUGH

Specialist Bathing

Gainsborough Bathrooms Ltd  
Gainsborough House  
Brickyard Road  
Aldridge WS9 8SR  
United Kingdom  
Tel: +44 (0)1527 400022

email: [service@gainsboroughbaths.com](mailto:service@gainsboroughbaths.com)  
[www.gainsboroughbaths.com](http://www.gainsboroughbaths.com)

## ***BERENO USER MANUAL***

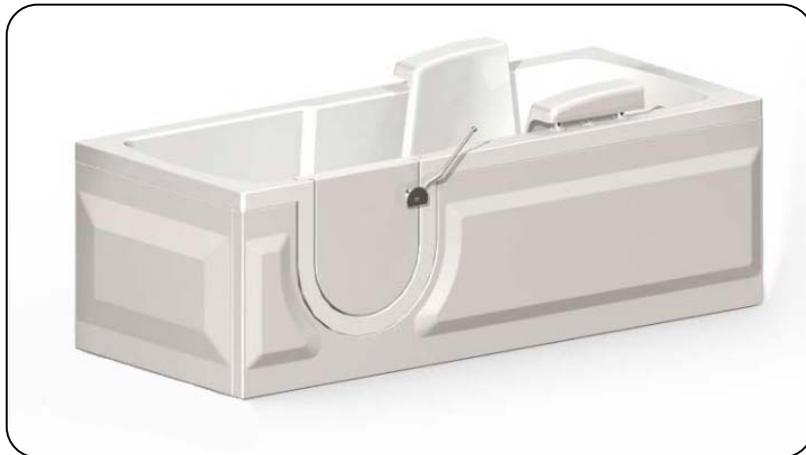

## **OPERATING INSTRUCTIONS**

Your bath is simple to use and maintain. However, there are some basic guidelines which should be followed to ensure you get the best out of your unit.

Please ensure that hands and objects are kept away from the underside of the seat when lowering into the bath.

To clean the unit, hot soapy water should be sufficient. Scouring powder and abrasive cleaners must never be used.

### ***To operate the unit, please follow these simple instructions:-***

The push buttons used to operate the bath seat are completely safe to use with wet hands.

The bath seat should be lifted from its parked position in the base of the bath using the up button. Once the seat is up it is recommended that you enter the bath and close the door.

Make yourself comfortable on the seat and commence filling the bath with water. Please note that the bath should not be filled to its maximum capacity, as when the user enters the bath, the water level will rise.

In the event of the bath being overfilled, water may either drain naturally through the overflow, or the plug should be temporarily removed to allow a sufficient amount of water to be drained away. Once the bath has been filled, the seat should then be lowered to the base of the bath.

Once you have finished bathing it may be emptied in the conventional manner.

The door must not be opened until the bath is completely empty of water.

### ***Removing the Seat***

To remove the seat, simply depress the two buttons located on the top of the yoke simultaneously, and at the same time lift the seat.

### **HELPFUL HINTS**

Make sure the bathroom is warm before you decide to have your bath. In this way you will be able to enjoy your bath fully.

A small towel draped across the shoulders whilst the bath is filling is also a good way to ensure you are fully relaxed.

We suggest that, as the bath is filling, you wash yourself working from the toes upwards so that as the water rises you are rinsed off automatically!

Keep a towel handy so that you can dry yourself as the bath empties.

## **Care**

The bath should be wiped down, preferably whilst the bath is draining, with a soft, damp soapy cloth. On no account should abrasive cleaners or bleaches be used.

Particular attention should be paid to the cleaning of the door seal, as the slightest amount of debris can cause the seal to leak.

Plated fittings, such as the handle and hinges, must only be wiped with a soft, damp soapy cloth.

A weekly wipe around the seal with a soft, damp soapy cloth is recommended to ensure it remains serviceable.

Chalk mark deposits from the water or mild scratches may be removed from the bath surface with a mild cutting fluid such as 'T' Cut or Brasso. This should be applied to the affected area with a soft cloth and rubbed vigorously until a satisfactory finish is effected. The shine to the area may then be brought back using a wax-based polish.

The shafts connecting the seat to the bath should be cleaned weekly using a silicone-based furniture polish. (i.e. Mr. Sheen). Please do not use wax-based products. When not in use, the seat should be left in the down position inside the bath.

## **IMPORTANT**

When not in use the door should be left in the closed position, although not in the fully latched closed position. This will prevent the hinges being strained whilst stopping the seal becoming compressed over time.

**ON NO ACCOUNT SHOULD PRESSURE BE PUT ON THE DOOR WHILST IN THE OPEN POSITION**

***PLEASE NOTE THERE ARE NO USER SERVICEABLE ITEMS ON THIS UNIT***

[service@gainsboroughbaths.com](mailto:service@gainsboroughbaths.com)